(Abb.11)

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Fabricating a dentobase Model - Positioning the Impression

5. All areas of the impression NOT required for fabricating the dental prosthesis can be trimmed! Trim the vestibular and lingual border areas of the impression in a funnel shape so that an adequately stable dental arch can be made. Trim the palatal region flat at benchtop level (Fig.11). Defect points in the impression are blocked out with dentobase putty (Prod. No. 39011).

Tip: Tough polyester impression compounds can be trimmed more easily by heating the tip of the knife slightly.

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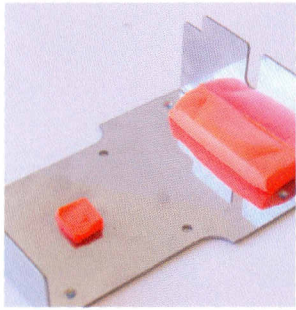
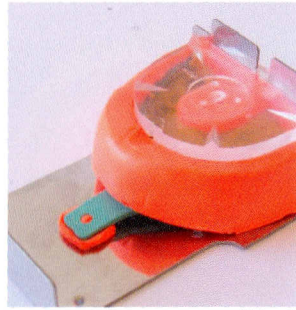
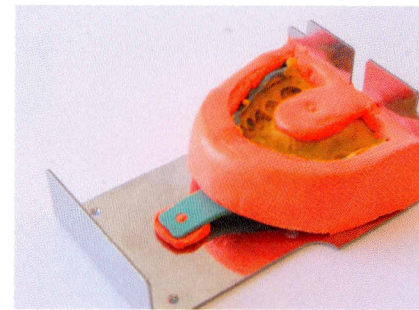
- 6.** Using some dentobase putty (Fig. 12) and the transparent dentobase alignment plate (Fig. 13), position the impression on the dentobase impression tray (Prod. No. 39041) to ensure an optimal centre line, vestibular dimensions and occlusal plane in the later dental arch.
- 7.** By “manipulating” the putty, you can now finalise the position and alignment of the impression. (Fig. 14)
- 8.** The impression is now finally fixed to the impression tray. (Fig. 15)

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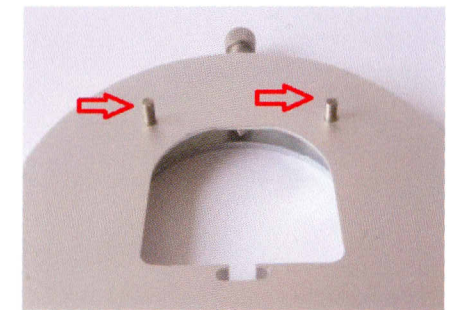
Preparation of the dentobase Universal Plate Holder

- 9.** Pick up the dentobase Universal Plate Holder (Prod. No. 39042), find a flat side made from slightly shiny steel and a matte silver aluminium side with two metal pins sticking out. (Fig. 16) This is the top side. The bottom side is made of steel so that the drill’s magnetic table can be fixed securely to the plate holder during the drilling process. (Note: As soon as the drilling process is started, a magnetic field is generated on the red magnetic table, only while drilling processes last. This keeps the plate holder in the correct position and ensures that it cannot slip.)

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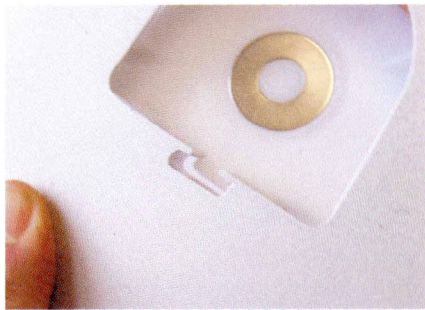
english

10. Take the required dentobase base plate (large or small) and insert the plate into the universal plate holder via the dovetail(Fig. 17) in such a way that split cast wedges and the retention inlay are visible from above. (Fig.18)

11. Now fix the base plate manually and gently in the holder with the knurled screw on the rear edge of the plate holder. (Fig.19)

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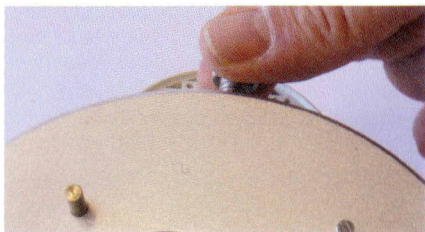
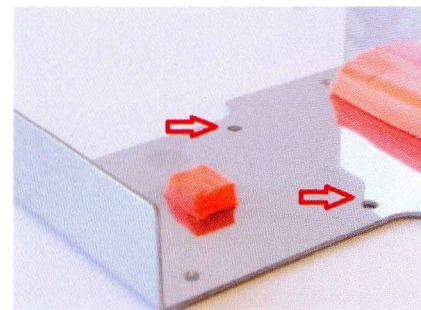
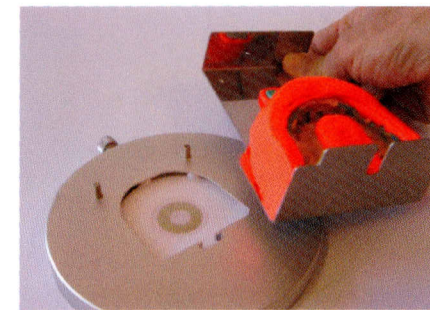
english

12. If you look closely at the impression tray, you will notice two punched holes around about the region of the 3rd canines. (Fig.20) These holes exactly fit the metal pins on the Universal Plate Holder. Now place the impression tray on the plate holder so that the impression lies precisely above the dentobase base plate. (Fig.21)

13. The impression tray is thus finally fixed to the Universal Plate Holder. (Fig.22)

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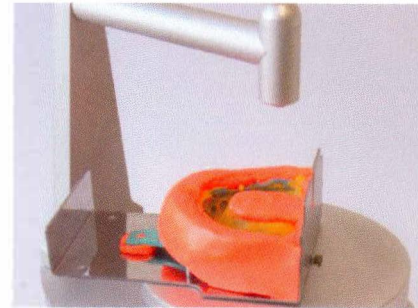
16. Now place this complete device on the magnetic table of the pin drill (Fig.25).

17. Use this laser pointer in the impression to mark the position of the pins in your later dental arch. The spot illuminated by the pointer will be drilled into the dentobase base plate from below (Fig.26). Follow the same procedure with each removable segment. **Important:** 2 pins per segment! Because it important to also saw between “1 and 1”, this region must be adequately “pinned” as well. **Tip: Please ensure that you only drill in the dental arch region, because the “retention inlay” region will cause the drill to break.**

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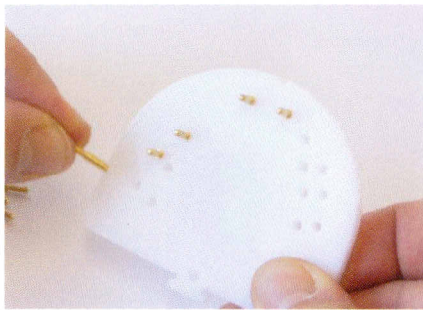
Placing Pins in the Drilled Base Plate

18. When all holes have been drilled, loosen the base plate from the base holder and insert a dentobase pin in each hole (Prod. No.: 39011) (Fig.27). The conical part of the pin goes in the hole. The pin's retention region sticks out and remains visible. *Tip: You can control the pins' friction a little by inserting them with greater or less effort.*

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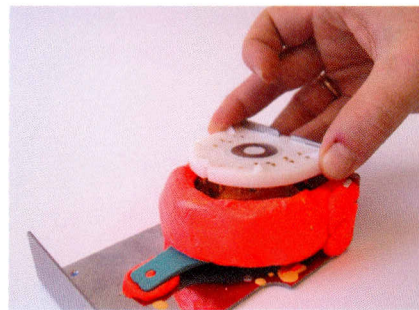
19. Tip: To ensure that none of the inserted pins are overlying the impression in any way, place the pinned base plate without plaster "in situ" (Fig. 28) via the dovetail on the base plate and the associated impression tray opening. Now the pinned base plate lies flat on the putty. If that is not the case, one of the pins is unexpectedly overlying the impression. The height can now be corrected with the lateral putty fixation. It is often enough to just shorten the corresponding pin with pliers.

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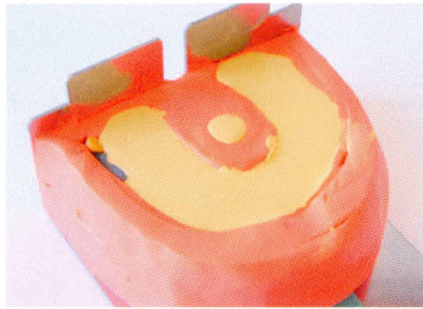
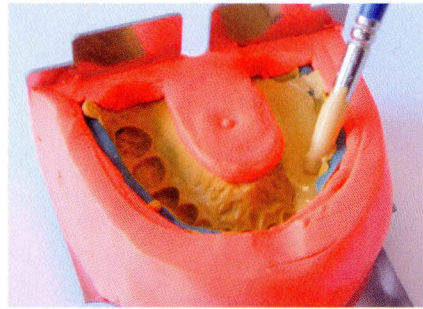
Mixing the Plaster and Pouring Out the Impression

20. Now mix the high-strength dental stone (esthetic-base® gold) correctly. Handle your impression appropriately for its type (reduce the surface tension of silicone, wet polyether with water) and carefully fill the trimmed impression up to the top margin, making sure that there are no bubbles (Fig. 29). *Tip: For safety, insert a thin instrument (probe) for bubble-free pouring out in "difficult" areas. (Fig.30)*

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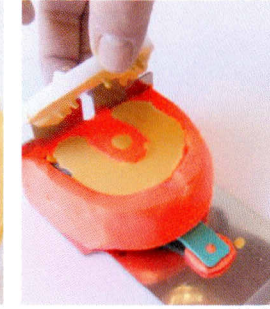
21. Now enclose the base plate around the pins in the region of the dental arch with plaster. (Fig.31) *Tip: If you briefly inject the base plates with a surfactant (e.g. coppie flux), the plaster runs much more easily between the pins.*

22. Now carefully lower the base plate through the dovetail (as already mentioned in Para 19) into the impression filled with plaster and thus join the base plate and impression together. (Fig.32) A little plaster oozes out of the impression all round. (Fig.33)

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Demoulding and Sawing

23. After the plaster has fully set (see technical data of plaster), loosen the impression with plaster from the impression tray (Fig.34) and carefully demould the dental arch. (Fig.35)

24. Gently tap into the oral region of the base plate with a small horn hammer to make sure only the dental arch comes loose from the plate. (Fig.36+37)

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25. Now work on the vestibular and oral regions of the dental arch with a roll of sandpaper or dental arch cutter. (Fig.38)

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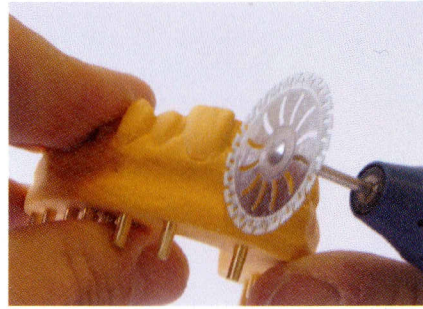
english

26. If you now reposition the dental arch on the associated base plate, you will notice that the dental arch does not fit! 1mm gap between dental arch and base plate (Fig.39) That is due to the expansion of the dental arch plaster. As soon as you have sawed the dental arch once (between "1 and 1" (Fig.40) is best), it will fit on the base plate without any gaps. (Fig.41)

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FERTIG (Abb. 43)

27. Now just saw segments (Fig.42) out and "prep".

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